




#### Guidance for chair with four legs or castors

1. Locate a link arm over each raising unit (2), making sure the lip faces up on one and down on the other as shown in (3) below
2. Set out as shown in (1) below, with loose wing nuts
3. Place linked raisers on the floor, as close as possible to each leg or castor
4. Adjust arm angles as required
5. Place furniture on raising units. Tip furniture whenever possible, forwards first, rather than lifting. Repeat by tipping backwards to fit front raisers.
6. Pull link arms together, allowing space for the users feet. Spreader bar should run "arm to arm", to avoid a hazard at front of chair.
7. Tighten the wing nuts to secure.
8. Ensure chair is stable on the raisers before use. Product to be used in static mode only.


(1)



(2)

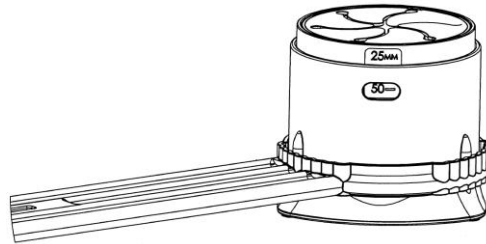

(3)

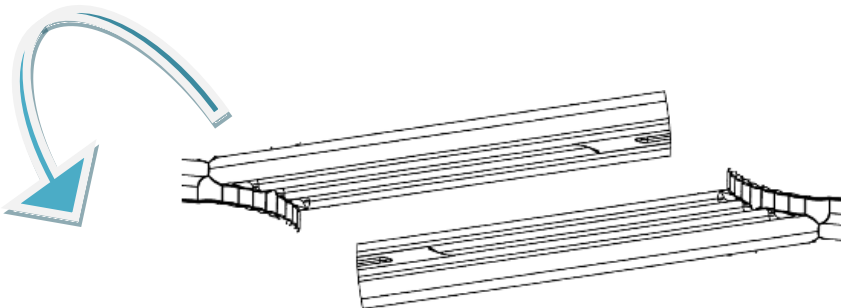

FM 522733



## User Guide



## Suregrip Link Arms & Spreader Bar



Article: 60571 – Two Link Arms  
Article: 60572 - Single Spreader Bar

[www.gordonellis.com](http://www.gordonellis.com)  
[enquires@gordonellis.co.uk](mailto:enquires@gordonellis.co.uk)

TRENT LANE  
CASTLE DONINGTON  
DERBY DE74 2PY  
ENGLAND

TEL: (01332) 810504 INT: + 44 1332 810504  
FAX: (01332) 850830 INT: + 44 1332 850830  
Company registered in England: No. 231734  
VAT No. GB 428 4666 26



NO TOOLS  
NEEDED