INSTRUCTIONS FOR USE MODE D'EMPLOI GEBRAUCHSANWEISUNG

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EN ONLY USE SUREGRIP RAISERS UNDER STATIC FURNITURE. WEIGHT OF FURNITURE SHOULD BE EVENLY DISTRIBUTED. IF KNOCKED CHECK TO ENSURE STABILITY. CHECK FOR STABILITY BEFORE EACH USE. ALWAYS RAISE FURNITURE LEGS BY THE SAME HEIGHT. DO NOT USE ON UNEVEN SURFACES.

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EN

If using “Raisers” on a hard floor fit the supplied three adhesive feet.

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If furniture feet are 8.5cm or more use “Block Adapter” component, **follow A overleaf**. If furniture feet are less than 8.5cm, or have casters **follow B overleaf**.

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To change height of “Raisers” place hand into base and turn to desired height. **Ensure all “Raisers” are the same height.**

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If using “Link Arms + Spreader Bar”, first follow A or B, and then **C overleaf**. Ensure “Spreader Bar” is across the width of the furniture **not front to back - as this may cause a trip hazard**.

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LANGHAM SUREGRIP®

FURNITURE RAISERS
LINK ARMS + SPREADER BAR

Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.

SEE REVERSE FOR FITTING INSTRUCTIONS
AVAILABLE IN THE FOLLOWING LANGUAGES:
EN FR DE ES NL

MAX
700 Kg
1540 lbs

MAXIMUM LOAD
PER SET OF 4

MAXIMALES GEWICHT
PRO 4ER-SET

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MAXIMAAL GEWICHT
PER SET VAN 4

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EN ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE MANUFACTURER - CONTACT DETAILS BELOW

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Your SUREGRIP® RAISERS were specifically developed by our UK Design Team to make furniture raising as simple as possible, and with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

FURNITURE RAISER

60570

LINK ARM

60571

SPREADER BAR

60572

BLOCK ADAPTER
WITH ADHESIVE PAD

DESIGNED IN THE UK

BHTA
British Healthcare Trade Association

5
PP

RECYCLABLE

WASTE

GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY, ENGLAND DE74 2AT

MADE IN CHINA

MD

IFU-REF 60570/0520-1

PART REF 60570 + 60571 + 60572

AUTHORISED REPRESENTATIVE
DETAILS CAN BE FOUND ONLINE AT:
GordonEllisHealthcare.co.uk/AuthorisedRep