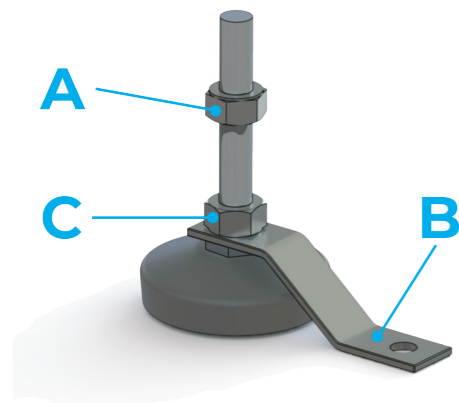
## SECURITY FIXING KIT

**Tools Required:**  **10 mm spanner (supplied\*)**  
 \*in Security Fixing Kit

1. Fit the angled bar **(B)** onto the foot, screw the nut **(C)** tightly against it.

2. Refit the feet to the step ensuring the nut **(A)** is tight against the Step, or base of the Extending Leg (if using).

Please read these instructions, along with the instructions on the reverse.

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# BIG FOOT® HALF STEP HANDLE

Welcome to your new Gordon Ellis product.  
 We hope that it proves a useful aid in your daily life.

**SEE INSIDE AND REVERSE FOR FITTING INSTRUCTIONS**  
 AVAILABLE IN THE FOLLOWING LANGUAGES:  
 EN FR DE ES NL PT DK EL

**HANDLES FOR SUPPORT ONLY.**    **POIGNÉES POUR LE SOUTIEN SEULEMENT.**  
**GRIFFE NUR ZUR UNTERSTÜTZUNG.**    **ASAS SOLO PARA APOYO.**  
**HANDGREPEN ALLEEN TER ONDERSTEUNING.**    **ALÇAS APENAS PARA SUPORTE.**  
**KUN TIL STØTTE.**    **ΛΑΒΕΣ ΜΟΝΟ ΓΙΑ ΥΠΟΣΤΗΡΙΞΗ.**

**EN** ONLY TO BE USED WITH THE BIG FOOT HALF STEP (66111) - PLEASE READ THESE INSTRUCTIONS ALONG WITH THE STEP INSTRUCTIONS BEFORE USE.

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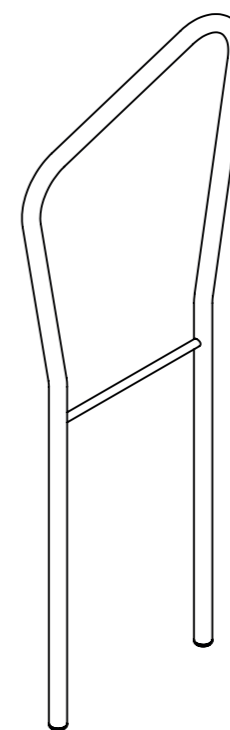
Your **BIG FOOT HALF STEP HANDLE** was specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

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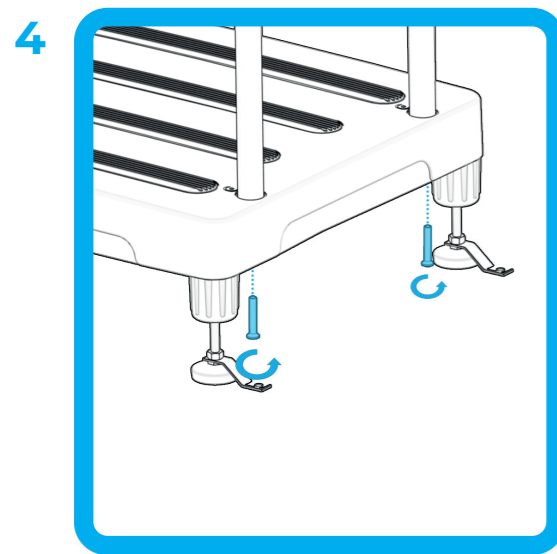
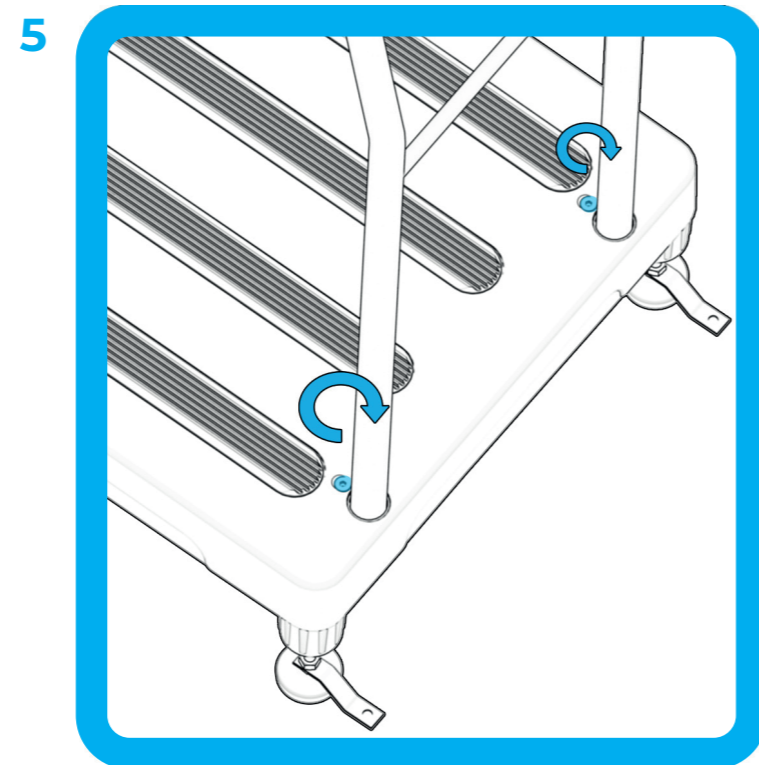
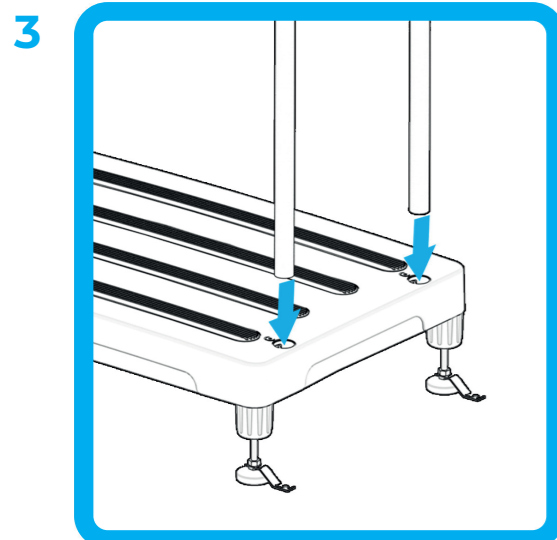
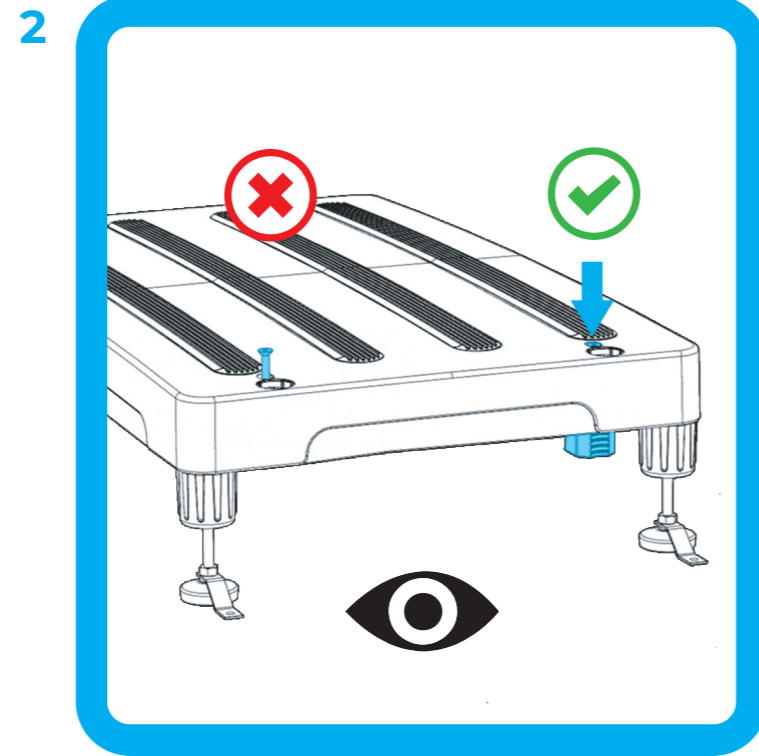
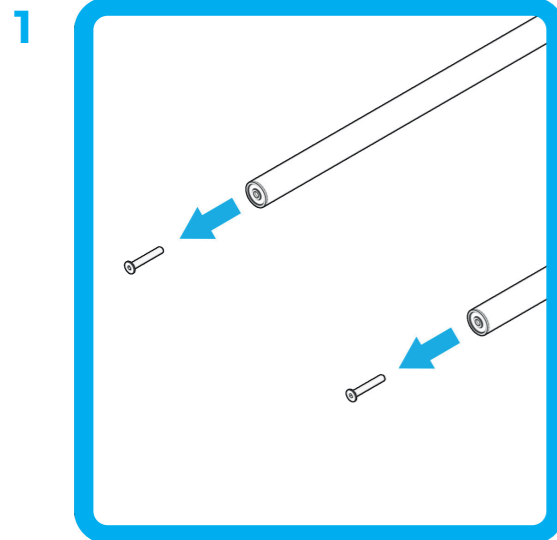
**EN** THE SUPPORT HANDLE IS DESIGNED TO ASSIST PEOPLE WHO ARE UNSTEADY ON THEIR FEET TO ACHIEVE A MORE SECURE TRANSFER USING THE STEP. IF REQUIRED A SUPPORT HANDLE CAN BE FITTED TO BOTH SIDES OF THE STEP. IT IS TO BE USED ONLY WHEN FITTED TO A BIGFOOT HALF STEP. INSTALL THE SECURITY FIXING KIT; DO NOT USE THE SUPPORT HANDLE WITHOUT INSTALLING THE SECURITY FIXING KIT. PLEASE REFER TO THE INSTALLATION GUIDE SHOWN BELOW. 1. TO FIT; REMOVE THE TWO COVERS AT THE SIDE OF THE STEP, UNSCREW TWO BOLTS FROM THE BASE OF HANDLE, INSERT INTO OPENINGS, RE-FIT SCREWS. 2. BEFORE USE CHECK THE STEP IS SECURE. 3. TREAD CENTRALLY ONTO THE STEP SURFACE AVOIDING STEPPING ON TO THE CORNERS OR ALONG THE EDGES OF THE STEP, DO NOT LEAN BACKWARDS AS THIS IS LESS STABLE FOR THE USER. IT IS THE RESPONSIBILITY OF THE OWNER/ USER TO CARRY OUT AN INITIAL AND REGULAR LOCAL RISK ASSESSMENTS. CHECK STABILITY BEFORE USE. PERIODICALLY CHECK ALL FIXINGS FOR TIGHTNESS. CHILD AND TODDLER USE UNDER SUPERVISION ONLY. HANDLES ARE FOR SUPPORT ONLY AND ARE NOT LOAD BEARING TO THE SAME WEIGHT AS THE STEP.

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