

## INSTRUCTIONS FOR USE

### STEP 9

ADJUST STRAPS TO FIT YOUR CHILD: SIT YOUR CHILD ON THE SEAT AND MAKE A NOTE OF THE HEIGHT OF THEIR SHOULDERS AND EXTREMITIES OF HIPS ON THE SEAT BACK. REMOVE YOUR CHILD AND LOCATE THE MOST APPROPRIATE STRAP HOLES IN THE SEAT BACK, AT THE TOP AND BOTTOM, AND ALSO IDENTIFY THE BEST OF THE THREE HOLES ON THE VERY FRONT OF THE SEAT PAD.


**9.1.** IF YOU NEED TO ADJUST THE STRAPS AT THE BACK, TURN THE STRAP ATTACHMENT ON THE BACK OF THE SEAT UNTIL IT IS CLEAR OF THE HOLE. **REMEMBER TO TURN IT BACK AFTERWARDS.**

**9.2.** YOU CAN NOW PUSH THE STRAP, INCLUDING THE BUCKLE, ALL THE WAY THROUGH THE BACK.

**9.3.** FEED IT BACK THROUGH THE HOLE WHICH IS AT THE BEST HEIGHT. NOW FEED IT THROUGH THE MOST SUITABLE HOLE AT THE BOTTOM OF THE SEAT (PART 6). CLICK TOGETHER BEHIND THE SEAT, UNDER THE REAR SLATTED SEAT (PART 4).

**MAKE SURE THAT THE STRAP IS NOT TWISTED.**

**9.4.** TO FINISH FITTING THE HARNESS, ATTACH THE FIFTH POINT OF THE HARNESS THROUGH THE MOST SUITABLE HOLE, USING PART 11. FEED THE BOLT (PART 11) THROUGH THE SEAT (PART 3) AND TIGHTEN THE NUT UNDERNEATH USING AN 8MM SPANNER.

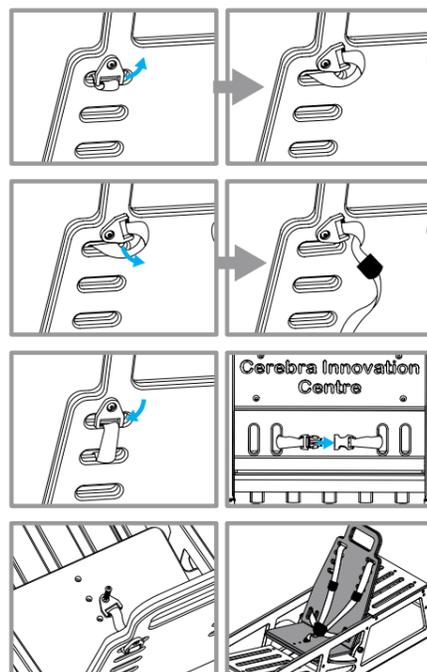

## ⚠ SAFETY WARNINGS ⚠

- THIS SLEDGE IS INTENDED FOR TRANSPORTATION ACROSS FLAT GROUND NOT DOWNHILL USE.
- ALWAYS TO BE USED UNDER THE SUPERVISION OF A RESPONSIBLE ADULT.
- ANY PERSON TRAVELLING ON THE SLEDGE SHOULD WEAR A HELMET AND SUITABLE PROTECTIVE CLOTHING.
  - WE RECOMMEND A HELMET TESTED TO BS EN 1078:1997 OR BS EN 1077:2007 SUITABLE FOR SNOW SPORTS.
- THE RESPONSIBLE ADULT SHOULD ALWAYS HOLD ONTO THE ROPE AT THE FRONT, EVEN WHILE THE SLEDGE IS NOT IN MOTION, TO PREVENT THE SLEDGE FROM SLIDING AWAY.
- IF RIDING ON THE REAR SEAT, PLEASE SECURE THE LEASH TO YOUR WRIST. SHOULD YOU FALL OFF, THIS WILL PREVENT THE SLEDGE FROM SLIDING OUT OF CONTROL.
- DO NOT STAND OR ATTEMPT TO RIDE WHILST STANDING ON THE SLEDGE.
- NEVER LEAVE THE SLEDGE UNATTENDED IN THE SNOW.
- NOT INTENDED FOR PROLONGED USE.
- THE SEAT IS IN NO WAY INTENDED AS A POSTURAL SUPPORT SEAT. IF IN DOUBT, PLEASE ASK YOUR MEDICAL PROFESSIONAL.
- THE STRAPS ARE SUPPLIED TO HELP HOLD YOUR CHILD IN THE SEAT, THEY ARE IN NO WAY INTENDED FOR SAFETY OR TO PREVENT INJURY DURING IMPACT OR FOR POSTURAL SUPPORT. PLEASE CONSULT YOUR MEDICAL PROFESSIONAL REGARDING USE OF THE STRAPS.
- BE AWARE OF THE COLD. WRAP YOUR CHILD UP WARM – REMEMBER THEY ARE NOT EXERTING MUCH ENERGY TO HELP KEEP THEM WARM.
- WEAR SUITABLE FOOTWEAR TO ENSURE YOUR CONTROL WHILST PULLING THE SLEDGE.
- ADHERE TO THE ASSEMBLY METHOD AND INSTRUCTIONS SUPPLIED.
- IF YOU INTEND TO MAKE ANY ALTERATIONS TO THE SLEDGE PLEASE DO SO CAREFULLY AND REMEMBER NOT TO LEAVE ANY SHARP EDGES, OR MAKE ALTERATIONS IN SUCH A WAY THAT MIGHT CAUSE INJURY E.G. USE NUTS AND BOLTS RATHER THAN SCREWS, SO THAT IF YOUR ALTERATION FAILS IT WILL NOT REVEAL A SHARP OBJECT.





# CEREBRA PAEDIATRIC SLEDGE

WELCOME TO YOUR NEW GORDON ELLIS PRODUCT.  
WE HOPE THAT IT PROVES A USEFUL AID IN YOUR DAILY LIFE.



SEE INSIDE AND REVERSE FOR FITTING  
INSTRUCTIONS  
AVAILABLE IN THE FOLLOWING LANGUAGES: EN

YOUR CEREBRA PAEDIATRIC SLEDGE WAS SPECIFICALLY DEVELOPED BY OUR UK DESIGN TEAM WITH THE INTENTION OF COMPENSATING FOR AN INJURY, OR OVERCOMING A TEMPORARY OR PERMANENT DISABILITY. HOPEFULLY MAKING DAILY LIVING THAT LITTLE BIT EASIER.

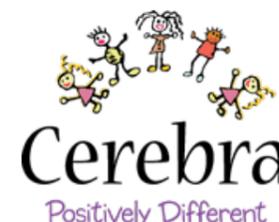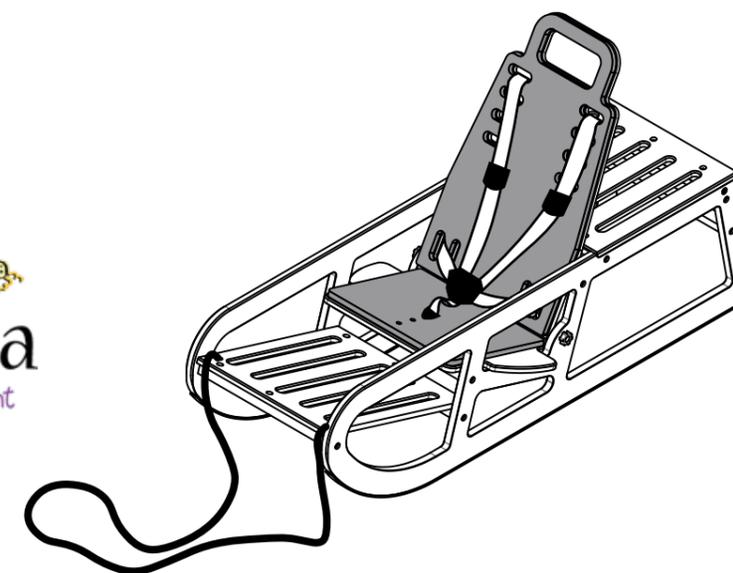

THE CEREBRA PAEDIATRIC SLEDGE HAS BEEN PRODUCED TO THE HIGHEST POSSIBLE STANDARD TO ENSURE THE SAFEST POSSIBLE BUILD QUALITY HOWEVER THERE ARE INHERENT RISKS ASSOCIATED WITH EQUIPMENT INTENDED FOR USE IN THE SNOW. AS SUCH PLEASE READ ANY SAFETY ADVICE CAREFULLY. NEITHER GORDON ELLIS & CO NOR CEREBRA CAN ACCEPT ANY LIABILITY FOR LOSS OR INJURY CAUSED DUE TO MISUSE OR ALTERCATIONS TO THE PRODUCT. MISUSE IN THIS INSTANCE IS CONSIDERED TO BE ANYTHING CONTRAVENING THE INSTRUCTIONS AND SAFETY WARNINGS OFFERED HEREIN.



PART REF 85-855

IFU-REF 85-855/1020-2

**GORDON ELLIS & CO.**  
TRENT LANE  
CASTLE DONINGTON  
DERBY. ENGLAND DE74 2AT

VIEW THE ENTIRE RANGE ONLINE AT  
[GORDONELLISHEALTHCARE.CO.UK](http://GORDONELLISHEALTHCARE.CO.UK)

[gordonellis.healthcare](https://www.facebook.com/gordonellis.healthcare)  
[@EllisHealthcare](https://twitter.com/EllisHealthcare)

VIEW THE ENTIRE RANGE ONLINE AT  
[GORDONELLISHEALTHCARE.CO.UK](http://GORDONELLISHEALTHCARE.CO.UK)

[gordonellis.healthcare](https://www.facebook.com/gordonellis.healthcare)  
[@EllisHealthcare](https://twitter.com/EllisHealthcare)


# ASSEMBLY INSTRUCTIONS

## TOOLS REQUIRED:

- TWO PEOPLE, ALTHOUGH IT IS POSSIBLE WITH ONE
- Mallet (OPTIONAL FOR STEP 1)
- SCREWDRIVER (STEP 2)
- 8mm SPANNER (STEP 9)

## PACKAGING CONTENTS:

- 2 x SLEDGE RUNNERS (PART 1 AND 2)
- 1 x PADDED SEAT (PART 3)
- 1 x PADDED SEAT BACK WITH HARNESS (PART 6)
- 1 x FOOT PLATE WITH ROPE (PART 7)
- 1 x REAR SLATTED SEAT WITH LEASH (PART 4)
- 1 x REAR STAY (PART 5)
- 18 x SETS OF NUTS, COLLARS AND BOLTS (PART 8A, B, C)
- 2 x SEAT ADJUSTER KNOBS (PART 9)
- 4 x WASHERS - 2 x METAL, 2 x PLASTIC (PART 10)
- 1 x BOLT, WASHER AND NUT SET (PART 11)
- 3mm HEX KEY

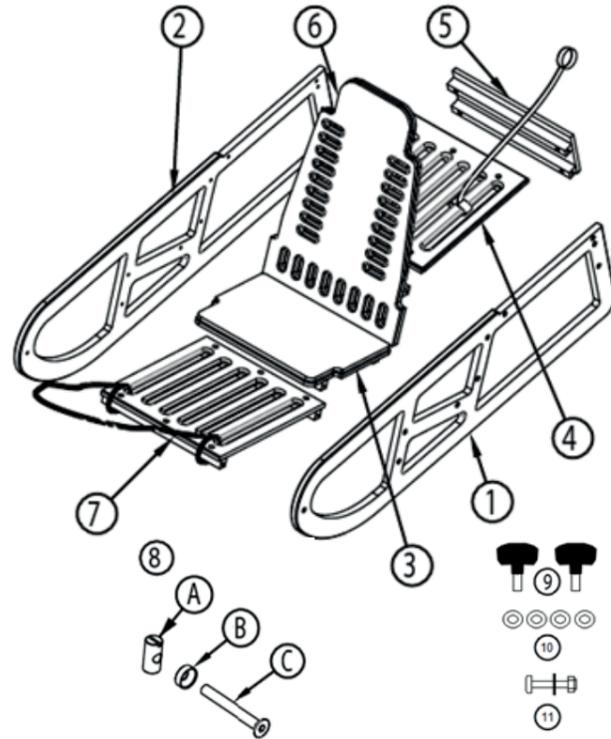

**i** IF ASSEMBLING INDOORS, BUILD THE SLEDGE ON ITS CARDBOARD PACKAGING TO PROTECT FLOORING.


## MAINTENANCE & STORAGE

- PLEASE CHECK THE TIGHTNESS OF NUTS AND BOLTS REGULARLY AND BEFORE USE.
- THE SLEDGE SHOULD BE STORED INSIDE, OUT OF THE ELEMENTS WHEN NOT IN USE. LEAVING IT OUTSIDE FOR PROLONGED PERIODS WILL DRAMATICALLY REDUCE ITS LIFE.
- THE SLEDGE IS CAREFULLY COATED WITH YACHT VARNISH LEAVING A WATER REPELLANT FINISH. LIKE ALL WOODEN PRODUCTS THE SLEDGE WILL REQUIRE CAREFUL MONITORING AND OCCASIONAL RE-VARNISHING USING A SIMILAR YACHT VARNISH.
- CLEAN WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.

## INSTRUCTIONS FOR USE

### STEP 1

TAKE THE 2 SLEDGE RUNNERS AND LAY FLAT ON THE FLOOR, WITH THE WIDER HOLES FACING UP. NOW TAKE ALL OF THE COLLARS (PART 8B) AND FIT ONE COLLAR INTO EACH HOLE. **DO NOT FIT COLLARS INTO THE 2 HOLES MARKED WITH AN X IN THE PICTURE.**

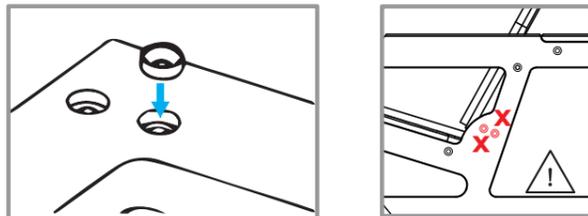


### STEP 2

TO INSERT BOLTS: FIRST, INSERT THE NUT (PART 8A) INTO THE HOLE AS SHOWN. IT IS IMPORTANT TO MAKE SURE THE HOLE ON PART 8A IS FACING OUTWARDS AND IS LINED UP TO RECEIVE THE BOLT (PART 8C). THE LINE ON THE TOP OF THE NUT (PART 8A) SHOULD BE FACING OUTWARDS.



**FIT THESE NUTS (PART 8A) INTO PARTS 3, 4, 5 AND 7 BEFORE CONTINUING.**

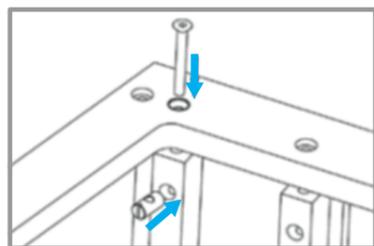


## INSTRUCTIONS FOR USE

### STEP 3

TAKE ONE SLEDGE RUNNER (PART 1 OR 2) AND ATTACH THE PADDED SEAT, REAR SEAT WITH LEASH AND FOOT PLATE WITH ROPE (PARTS 3, 4 AND 7) TO IT USING 6 SETS OF THE NUTS AND BOLTS (PARTS 8A AND 8C). **DO NOT FULLY TIGHTEN THE BOLTS YET.**



- **YOU WILL FIT THE SEAT BACK (PART 6) IN THE NEXT STEP.**
- **DO NOT FIT THE REAR STAY (PART 5) YET.**

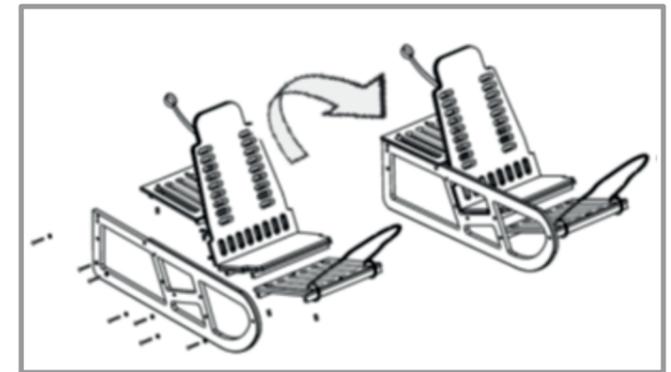


### STEP 4

ATTACH THE SEAT BACK (PART 6) USING ONLY THE HOLE MARKED WITH AN ARROW, USING 2 SETS OF THE NUTS AND BOLTS (PARTS 8A AND 8C), ONE SET PER SIDE.


**DO NOT PUT BOLTS IN THE HOLES MARKED WITH AN X.**

USE ONE OF THE TWO WHITE PLASTIC WASHERS (PART 10) BETWEEN THE SEAT BACK (PART 6) AND THE INSIDE OF THE SLEDGE RUNNER (PART 1 OR 2).

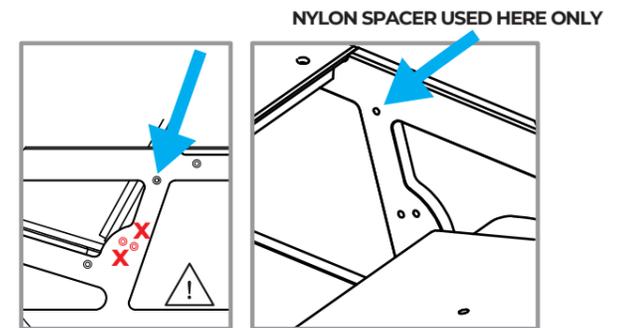


### STEP 5

TAKE THE SECOND SLEDGE RUNNER (PART 1 OR 2) AND REPEAT STEPS 3 AND 4 TO ATTACH ALL THE COMPONENTS IN THE SAME WAY USING ANOTHER 6 SETS OF NUTS AND BOLTS.

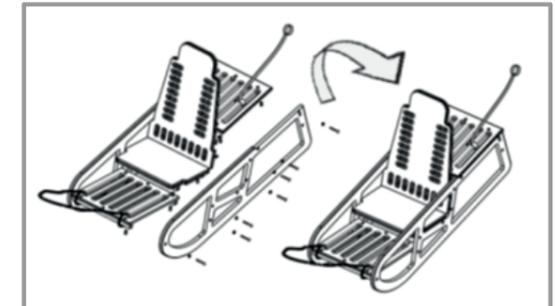

### STEP 6

TAKE THE REAR STAY (PART 5) AND ATTACH USING 4 SETS OF NUTS AND BOLTS.

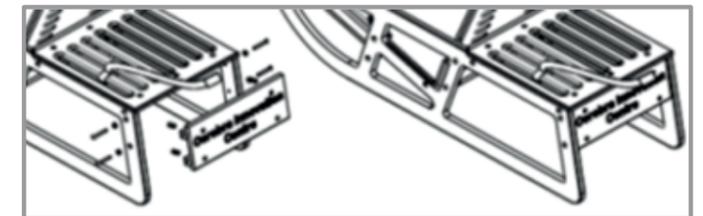

### STEP 7

TIGHTEN EACH BOLT IN TURN USING THE SUPPLIED ALLEN KEY TO A TIGHT FIT. WORK FROM FRONT TO BACK. ENSURE YOU HAVE CORRECTLY TIGHTENED ALL BOLTS AND THAT NONE HAVE BEEN OVERLOOKED.

### STEP 8

ADJUST THE SEAT TO SUIT YOUR CHILD: THE SEAT HAS A CHOICE OF TWO ANGLES OR FOLDS FLAT FOR TRANSPORT OR STORAGE. THE TWO SEAT ADJUSTER KNOBS (PART 9) HOLD THE SEAT BACK IN PLACE. USE ONE ON EITHER SIDE OF THE SLEDGE, IN POSITION A OR B, TO ADJUST THE ANGLE OF THE SEAT BACK.

**USE ONE OF THE METAL WASHERS (PART 10) BETWEEN THE KNOB AND THE SLEDGE RUNNER.**

