
SECURITY FIXING KIT

Tools Required:  **10 mm spanner (supplied*)**
 *in Security Fixing Kit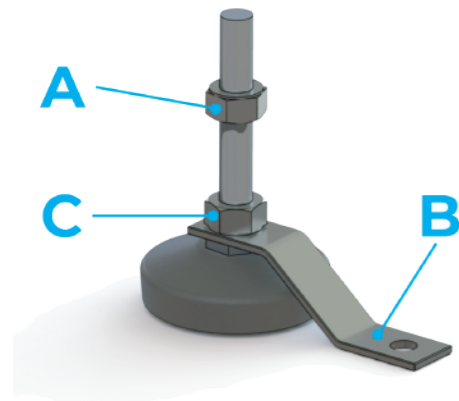

1. Fit the angled bar **(B)** onto the foot, screw the nut **(C)** tightly against it.
2. Refit the feet to the step ensuring the nut **(A)** is tight against the Step, or base of the Extending Leg (if using).

Please read these instructions, along with the instructions on the reverse.

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BIG FOOT® HALF STEP

Welcome to your new Gordon Ellis product.
 We hope that it proves a useful aid in your daily life.

SEE INSIDE AND REVERSE FOR FITTING INSTRUCTIONS
 AVAILABLE IN THE FOLLOWING LANGUAGES:
 EN FR DE ES NL PT DK EL

MAX 220 Kg 485 lbs	MAXIMUM USER WEIGHT	POIDS MAXIMUM DE L'UTILISATEUR	MAXIMUMGEWICHT VAN GEBRUIKER
MAXIMALES BENUTZGEREWICHT	PESO MÁXIMO DEL USUARIO	ΜΑΧ. ΒΡΥΓΕΡΒ/ΕΓΤ	ΑΝΩΤΑΤΟ ΟΡΙΟ ΒΑΡΟΥΣ ΤΟΥ ΧΡΗΣΤΗ
PESO MÁXIMO DE USO			

EN ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE MANUFACTURER - CONTACT DETAILS BELOW

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
Your **BIG FOOT® HALF STEP** was specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

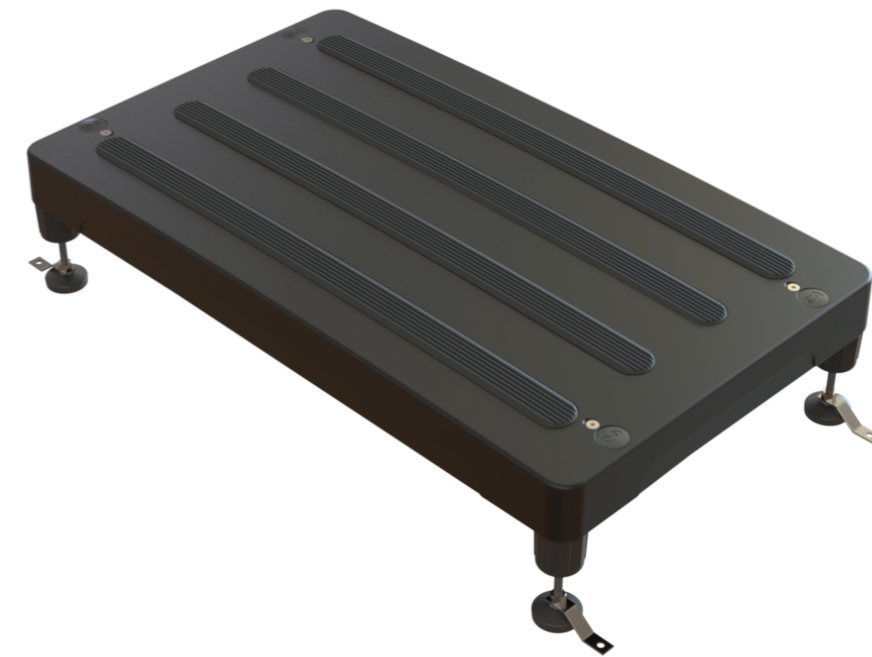DESIGNED AND MADE IN THE UK

GORDON ELLIS & CO.
 TRENT LANE
 CASTLE DONINGTON
 DERBY, ENGLAND DE74 2AT

AUTHORISED REPRESENTATIVE
 ADVENA LTD, TOWER BUSINESS CENTRE,
 2ND FLOOR, TOWER STREET, SWATAR, BKR
 4013 MALTA

MD





IFU-REF 6611/1020-2

PART REF 6611 + 6611/B

STEP - FEET & EXTENDING FEET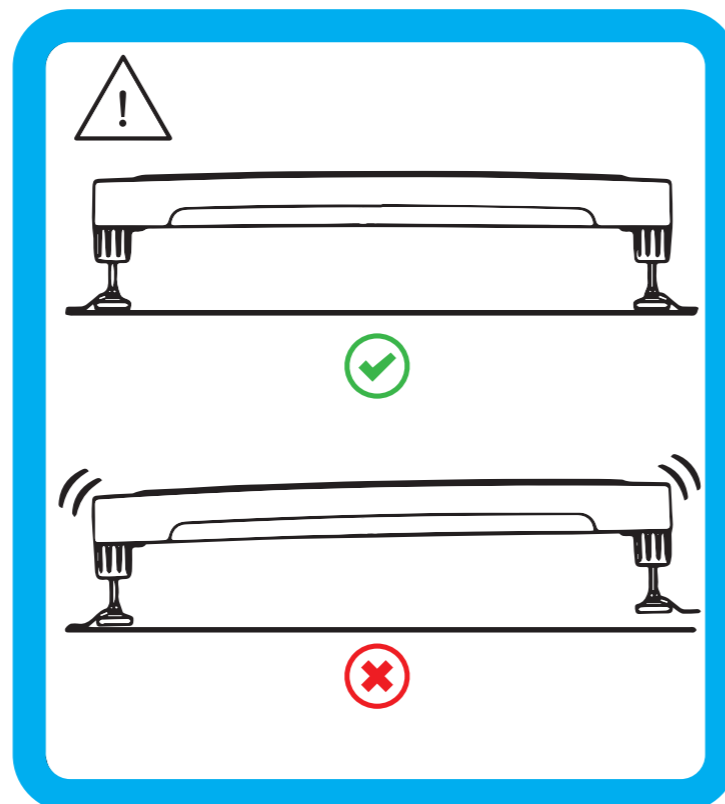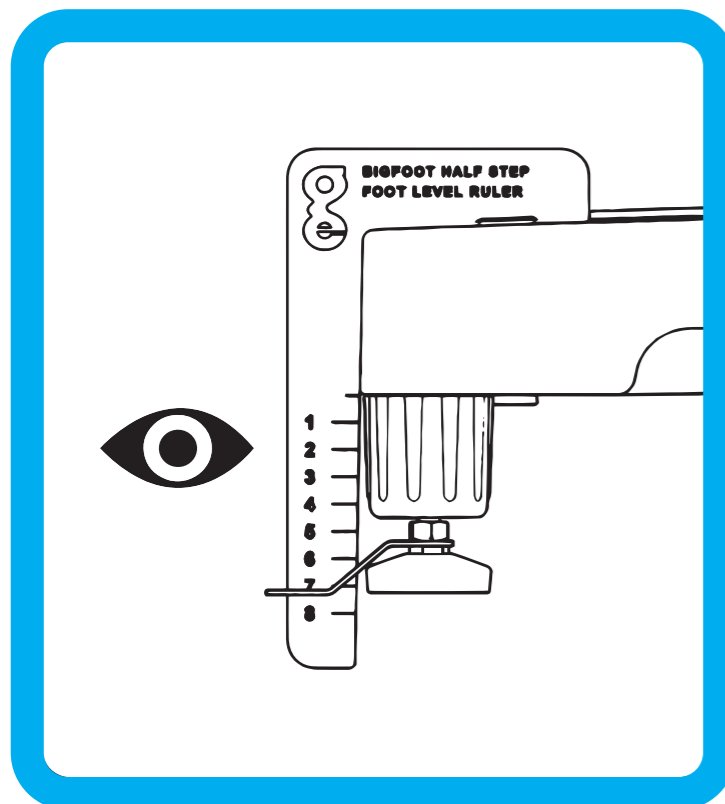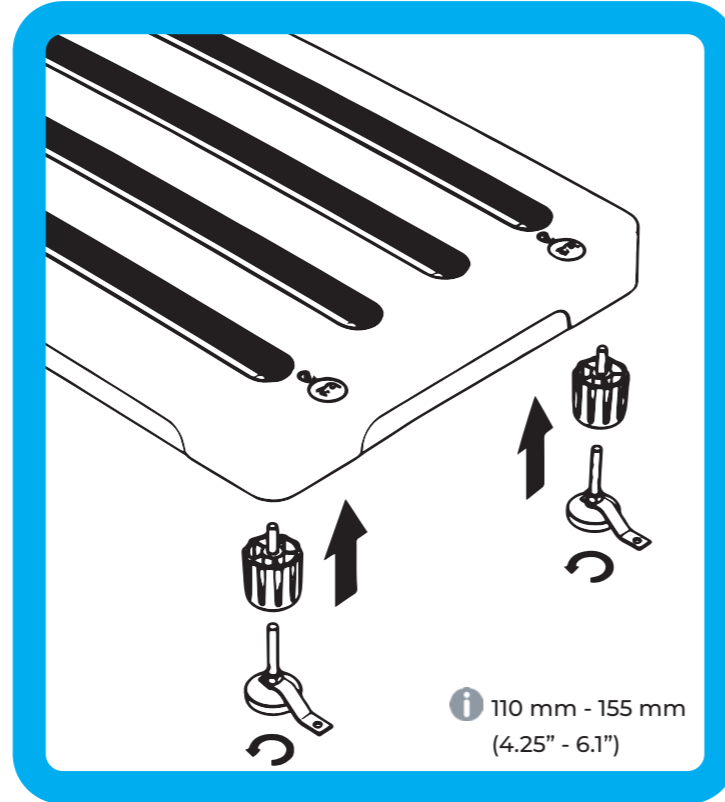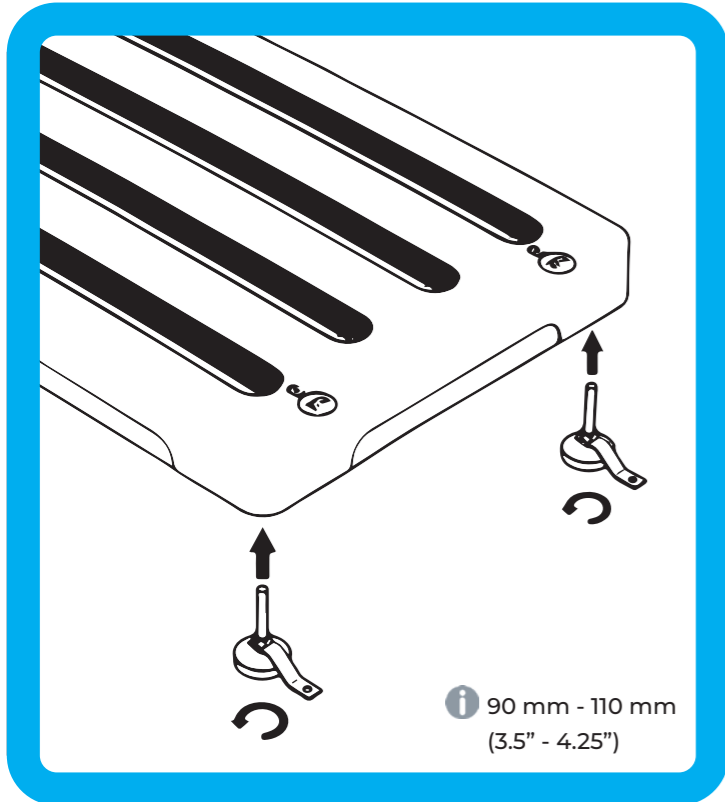

EN THE BIG FOOT HALF STEP IS DESIGNED TO ASSIST PEOPLE WHO ARE UNSTEADY ON THEIR FEET TO ACHIEVE A MORE SECURE TRANSFER. DO NOT USE THE STEP IN SLIPPERY CONDITIONS, FOR EXAMPLE IF COVERED IN ICE. IF FITTING THE SECURITY FIXING KIT PLEASE REFER TO THE INSTALLATION GUIDE ON THE OTHER SIDE OF THE PAGE. 1. SCREW THE FEET IN TO THE ADJUSTABLE LEGS. POSITION THE HALF-STEP ALONGSIDE THE BUILDING ENTRANCE OR THRESHOLD. THE LONGEST SIDE OF THE STEP SHOULD BE PUSHED UP AGAINST THE ENTRANCE OR OTHER STATIC SURFACE. 2. IF NECESSARY ADJUST THE HEIGHT OF THE STEP BY SCREWING THE SUPPLIED EXTENDER LEGS INTO THE STEP. FINE ADJUSTMENT TO LEVEL THE STEP IS DONE BY ADJUSTING THE FEET USING THE SUPPLIED SPANNER. THIS WILL ENABLE A SOUND AND LEVEL STEP TO BE SET. 3. BEFORE USE CHECK THE STEP IS SECURE. 4. TREAD CENTRALLY ONTO THE STEP SURFACE AVOIDING STEPPING ONTO THE CORNERS OR ALONG THE EDGES OF THE STEP, DO NOT LEAN BACKWARDS AS THIS IS LESS STABLE FOR THE USER. 5. THE HALF-STEP AND THE GROUND THE STEP IS PLACED ON MUST BE KEPT CLEAN TO AVOID SLIPPAGE. IT IS THE RESPONSIBILITY OF THE USER/ OWNER TO CARRY OUT AN INITIAL AND REGULAR LOCAL RISK ASSESSMENTS. CHECK STABILITY BEFORE USE. PERIODICALLY CHECK ALL FIXINGS FOR TIGHTNESS. CHILD AND TODDLER USE UNDER SUPERVISION ONLY. DO NOT USE WITH OTHER STEPS. NOT INTENDED FOR ANY OTHER USE. SUPPORT HANDLES ARE FOR SUPPORT ONLY AND NOT LOAD BEARING TO THE SAME WEIGHT AS THE STEP. CAN BE USED WITH HANDLES OR A SUITABLE GRAB BAR/ RAIL TO ACHIEVE A MORE SECURE TRANSFER.

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