

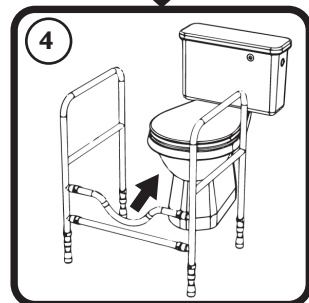
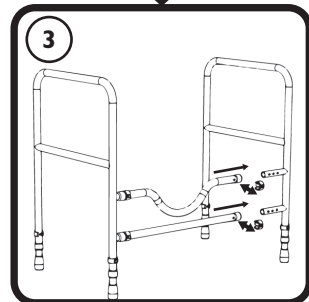
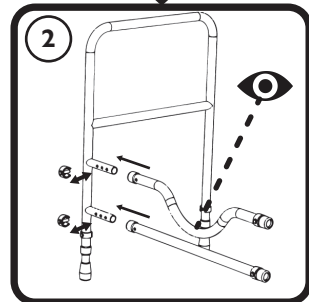
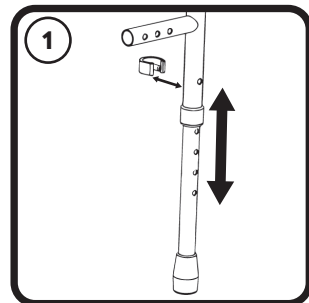
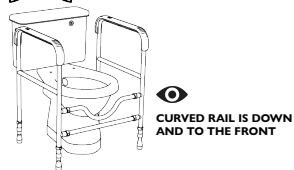
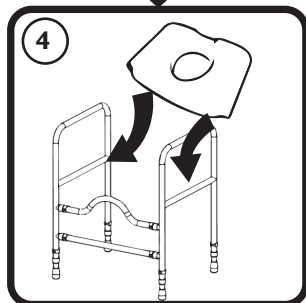
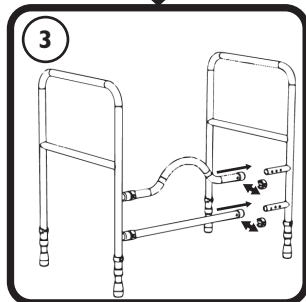
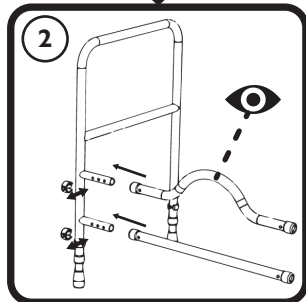
ci TOILET SURROUND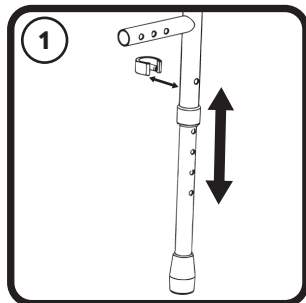

NOTE: ONE LEG FEATURES EXTRA HOLES FOR FINER HEIGHT ADJUSTMENT AND GREATER STABILITY.

Di TOILET AID

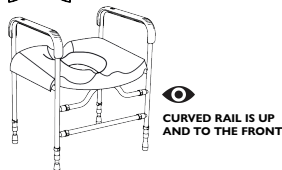
PRIMA® MULTIFRAME

TOILET SURROUND, TOILET FRAME & ARMS


Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.

Your **PRIMA® MULTIFRAME** was specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.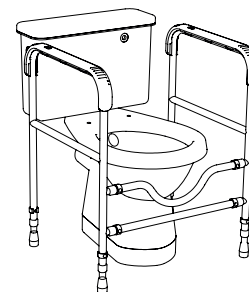


**TOILET
SURROUND**

65233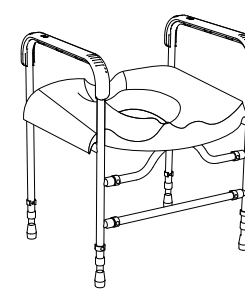

**TOILET
AID**

65240
65244

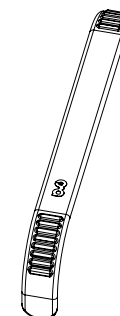ARM

61530
61531

DESIGNED
& MADE
IN THE UK

MD

MDPE

GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY, ENGLAND DE74 2AT

EC REP

AUTHORISED REPRESENTATIVE
ADVENA LTD, TOWER BUSINESS CENTRE,
2ND FLOOR, TOWER STREET, SWATAR,
BKR 4013 MALTA

i

IFU-REF 65233-0924-1


PART REF 65233 + 65240 + 65244 + 61530 + 61531

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

gordonellis.healthcare

@EllisHealthcare

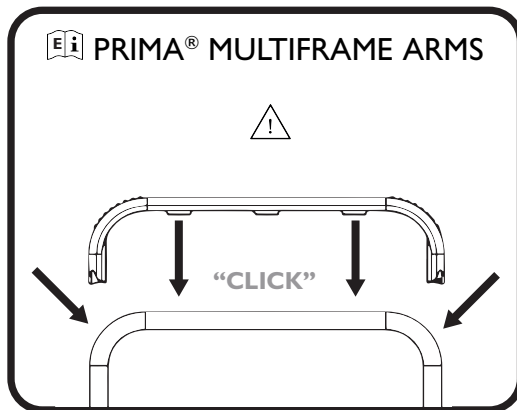**A****B****EN**

DESIGNED TO AID IN SITTING AND RAISING FROM THE SEAT. CAN BE USED AS A TOILET SURROUND OR TOILET FRAME. FOR STATIC USE ONLY. BEFORE USE, ENSURE STABILITY. ALWAYS GRIP THE HANDRAIL SECURELY. MAKE SURE: SPRING CLIPS PROTRUDE COMPLETELY THROUGH THE ADJUSTMENT HOLES; ALL LEGS ARE RAISED TO THE SAME HEIGHT. NOTE: ONE LEG FEATURES EXTRA HOLES FOR FINER HEIGHT ADJUSTMENT AND GREATER STABILITY.

A = MAXIMUM USER WEIGHT.
B = WIPE WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.

ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE EC REP (EU CUSTOMERS) OR THE MANUFACTURER.

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Triman - Indicates recycling and sorting rules (only relevant for France)

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